

Commode Wheelchairs User Manual

0808378, 0806777, 0808396, 0806063, 0810120, 0806053, 0808370



Commode Wheelchairs are delivered assembled & ready for use.

INSTRUCTIONS FOR USE

Commode Wheelchairs can be used by people with limited mobility inside the house.

Commode Wheelchairs **CANNOT** be used OUTSIDE the house.

Before using the Commode Wheelchair make sure that:

- The seat is correctly placed and secure in this position.
- The tank is correctly placed into the appropriate slot.
- There are no cracks or fractures in the frame.

MOBIAK A.E
Μ.Μπότσαρη 96-98, Χανιά-Κρήτης

MOBIAK S.A
Μ.ΜΠΟΤΣΑΡΙ 96-98 Chania-Crete, Greece
+30-28210-63222

www.mobiakcare.gr Email: mobiakcaresales@mobiak.gr



If trouble occurs in any of the above

DO NOT use the commode wheelchair until the problem is resolved .

FURTHER INSTRUCTIONS

- To provide the desired angle on the back of the cab (if it has reclining back) press the handle located under the handles.
- The wheelchairs have brake on the rear wheels which must be secured when the wheelchair does not move.
- When sitting or standing up from the wheelchair push the handles down, securing that it will stand still and not slip.
- DO NOT step on the foot rests to sit or get up, roll RISK. The footrests are used to place the user's feet when seated.
- When you get out of the walker must rely on both handles equally. Leaning only to one handle (without escort) can cause injury or damage the product.
- Regularly clean the wheelchair.
- Do not let the wheelchair wet.
- Do not let children near the wheelchair without supervision.

Warranty:

The product has a warranty period of one (1) year. The warranty covers factory defects and not damages from misuse/maintenance product.

MOBIAK A.E
Μ.Μπότσαρη 96-98, Χανιά-Κρήτης

MOBIAK S.A
Μ.ΜΠΟΤΣΑΡΙ 96-98 Chania-Crete, Greece
+30-28210-63222

www.mobiakcare.gr Email: mobiakcaresales@mobiak.gr

