

STANDARD KNEE WALKER INSTRUCTION

Before applying the brace, please read these instructions completely and carefully.

Correct application is vital to the proper function of the brace.

INDICATIONS:

Acute ankle sprain

Stress fractures of the lower leg

Stable delayed union or non-union fractures of the distal tibia and fibula

Achilles tendon repairs

CONTRAINDICATIONS:

Application of this device is recommended only when the fracture is demonstrably stable and there are acceptable limits of angular and rotational deformity. The determination of when to apply the braces rests strictly at the discretion of the treating physician.

This product was designed to complement the variety of medical treatments common to the above afflictions. The frequency and duration of use should be determined by your prescribing healthcare professional .



For first time application, loosen straps and remove liner from boot.



Place foot in liner and secure with contact closure. Fasten the foot flaps on the liner first, making sure they are snug. Wrap and fasten the leg portion of the liner, again making sure that it is snug from the bottom to top. Some trimming of the liner may be necessary to accomplish optimum fit



Spread the uprights using both hands and step into boot aligning uprights with midline of the ankle.



If uprights have plastic tube liner, remove them now and affix upright to liner.



Secure boot straps at the toes and working up the leg.

Foam pads are provided to alleviate the problem of excess rubbing and pads can be placed either inside the liner or between the liner and the uprights .

ΚΑΤΑΣΚΕΥΑΣΤΗΣ: ΜΟΒΙΑΚ Α.Ε./ΜΟΒΙΑΚ S.A
Μ.ΜΠΟΤΣΑΡΗ 96-98 ΤΚ. 73136- ΤΗΛ: 28210 63222, ΧΑΝΙΑ-ΚΡΗΤΗΣ
Μ.Μποτσάρι 96-98, 73136 Chania-Crete, Greece
Web site: <http://www.mobiakcare.gr>, e-mail: mobiakcaresales@mobiak.gr

